



CL-1000i

Chemiluminescence Immunoassay System

mindray
healthcare within reach

CL-1000i

Chemiluminescence Immunoassay System



Simple yet Rol



Enhanced Reaction System

4-phase magnetic separation with precisely controlled temperature
Consistent performance reliability with enhanced PMT detection system
Non-touch vortexer mixing for reaction solution without any contamination



Sample & Reagent Probe


Precisely controlled syringe motors offer high precision aspiration
Intelligent liquid level detection, clot detection, vertical and horizontal collision protection
Efficient interior & exterior probe wash with high pressure wash buffer
Programmable enhanced probe wash with detergent



Sample Handling


Samples loading and offloading continuously by sample racks
Unified sample racks with detachable adaptor
Rack status real-time indicating on lane numbers
STATs prioritized fast

oust

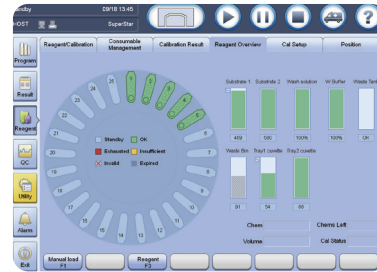

Intuitive Software Interface

Unified software interface, significantly reduces training effort
Smooth interchange by intuitive software navigation
Easy access to touch screen monitor with swing arm (optional)

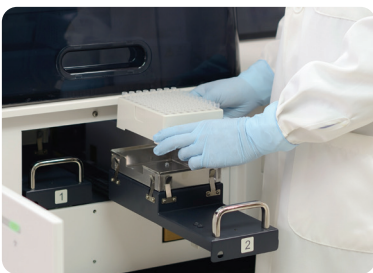

Consumables Management

Random access for cuvette loading without pause or stop
88*2 cuvettes in one batch, 200 cuvettes per waste bag
Cuvettes & waste bag status real-time indicating



Reagent & Substrate Handling

Easy for reagent & substrate handling, liquid reagents ready-to-use
2°C~8°C non-stop refrigerated reagent carousel featured 25 positions
Same reagent pack size with CL-2000i (50 tests/100 tests per pack)
Reagents stable on board for up to 28 days



Test Menu

Thyroid

FT3
FT4
T3
T4
TSH
Anti-TG
Anti-TPO
Thyroglobulin (TG)

Fertility

Total β -HCG
FSH
LH
PRL
E2
E3
TESTO
PROG
DHEA-S

Tumor Marker

CEA
AFP
CA125
CA15-3
CA19-9
FPSA
TPSA
FERR
NSE*
CYFRA 21-1*
CA72-4*

Infectious Disease

HBsAg*
Anti-HBs*
HBeAg*
Anti-HBe*
Anti-HBc*
HIV Ag/Ab Combo*
Anti-HCV*
Anti-TPPA (syphilis)*

Cardiac Marker

Myoglobin
Troponin I
BNP
CK-MB(mass)

Diabetes

Insulin
C-peptide

Bone Marker

PTH
Calcitonin
Vitamin D Total

Anemia

Folic Acid*
Vitamin B12*

Others

ACTH
Cortisol

*CE not Ready

CL-1000i

Chemiluminescence Immunoassay System

Technical Specifications

General Information

Throughput	Up to 120 tests/hour
Measurement Principle	Micron superparamagnetic particles platform with alkaline phosphatase (ALP) labeled reagents and AMPPD substrate

Sample Assembly

Sample Feeder	Continuous sample loading and offloading by sample racks
Sample Rack	10 positions per rack, same as BS-800/BS-2000/CL-2000i series
Sample Capacity	Up to 60 samples in one batch
Sample Probe	Steel probe, liquid level detection, clot detection, horizontal and vertical collision protection
Sample Volume	10 μ l~200 μ l, with increment of 1 μ l
Sample Dilution	Automatic dilution with 1:2 ~ 1:40

Reagent Assembly

Reagent Disk	25 reagent positions, continuous loading, real-time mixing for magnetic particles
Cooling System	2 $^{\circ}$ C~8 $^{\circ}$ C
Reagent Pack	50 tests per pack, 100 tests per pack
Reagent Scanning	Built-in/external bar code reader scanning
Reagent Probe	Steel probe, liquid level detection, bubble detection, horizontal and vertical collision protection
Reagent Volume	20 μ l~200 μ l, with increment of 1 μ l

Substrate System

Substrate Bottle	250/500 tests per bottle, bottom piercing
Substrate Loading	Capable of continuous loading during testing
Substrate Heating	Constant heating
Aspiration Volume	200 μ l

Magnetic Separation Unit

Separation Assembly	4-phase magnetic separation
Temperature	Accuracy: 37 \pm 0.3 $^{\circ}$ C, fluctuation: \pm 0.2 $^{\circ}$ C

Measurement and Reaction System

Detection Mode	Photon counting
Signal Detector	Photomultiplier (PMT) photon counter
PMT Calibration	LED reference module
Reaction Carousel	75 positions
Temperature	Accuracy: 37 \pm 0.3 $^{\circ}$ C, fluctuation: \pm 0.2 $^{\circ}$ C
Mixing Unit	Non-touch vortexer mixing

Cuvette Loading and Waste Collection

Cuvette	Disposable plastic cuvette
Cuvette Loading	88 cuvettes per tray, two trays can be loaded in one batch, continuous loading and offloading
Waste Collection	Waste container, 200 cuvettes per box

Working Conditions

Power Supply	110V/115V~, 60Hz; 220V-240V~, 50Hz; 220V/230V~, 60Hz
Input Power	2000VA
Operating Temperature	15 $^{\circ}$ C~30 $^{\circ}$ C
Relative Humidity	35%RH~80%RH, without condensation
Dimension	Main Unit: 1400mm*760mm*600mm (W*D*H) Weight: 250kg
Altitude Height	-400m~4000m

Mindray Building, Keji 12th Road South,
High-tech Industrial Park, Nanshan, Shenzhen 518057, P.R. China
Tel: +86 755 8188 8998 Fax: +86 755 26582680
E-mail: intl-market@mindray.com www.mindray.com

Mindray is listed on the NYSE under the symbol "MR"

mindray are registered trademarks or trademarks owned by Shenzhen Mindray Bio-medical Electronics Co., LTD.
© 2015 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved. Specifications subject to changes without prior notice.
P/N: ENG-CL1000i-21285x6-20151009

mindray